


116BB

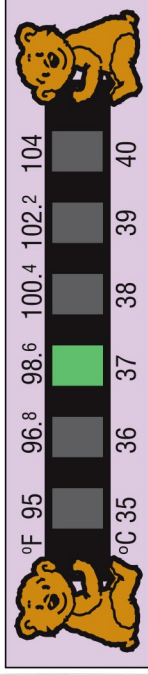

Thermometer size: 3.5"W x .75"H

Instructions for Use

How to Read

- Apply to center of forehead just above the eyebrows.
- Read thermometer when the boxes stop is still on the forehead.
- Green indicates the correct temperature. If green does not appear, temperature is between blue and tan.

Intended Use

- Measure human forehead surface temperature and monitor temperature trends

Accuracy

- ±1°F (0.6°C)

Liquid Crystal Forehead Thermometers are designed to be unbreakable noninvasive, quick reading devices that reliably indicate core body temperature by measuring the surface temperature of the forehead. Liquid Crystal Forehead Thermometers indirectly measure core body temperature by measuring the forehead temperature, whereas clinical thermometers (oral/rectal) directly measure core body temperature. Unusual Liquid Crystal Forehead Thermometer readings should be verified by a clinical thermometer.

Conforms to ASTM E-1061-01 For external use only.

LCR Hallcrest, 1911 Pickwick Lane, Glenview IL 60026 USA

ACT063 Rev04

Storage & Handling

- Store at 60 to 104°F in 15 to 95% RH
- Optimal storage at 60 to 80°F in 40 to 60% RH

Precautions

- Environmental temperatures might affect the performance of this product
- Product should be used indoors at normal room temperature
- Avoid exposure to UV light
- Clean after use with soap and water or rubbing alcohol. Do not soak.

Instructions for Use—The instructions for use are located on the inside of the folded paper insert

NOT MADE WITH NATURAL RUBBER LATEX

Part#: 116BB
Lot#: MXXXXX
Qty: 1 ea.

(01)20815236020999 (10)MXXXXX

FeverScan
CHILD & INFANT THERMOMETER

Imprint: 3"W x .75"H